

Overview

Useful For

Diagnosis of cystinuria

Genetics Test Information

Biochemical diagnosis of cystinuria. Measures cystine, lysine, ornithine, and arginine.

Special Instructions

- [Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens](#)

Method Name

Liquid Chromatography-Tandem Mass Spectrometry (LC-MS/MS)

NY State Available

Yes

Specimen

Specimen Type

Urine

Necessary Information

24-Hour volume is required.

Specimen Required

Supplies: Urine Tubes, 10 mL (T068)

Specimen Volume: 5 mL

Collection Instructions:

1. Collect before intravenous pyelogram.
2. Collect urine for 24 hours.
3. Add 20 mL of toluene as preservative at start of collection. If toluene is not available, refrigerate during collection.
4. Mix well before taking 5-mL aliquot.

Additional Information: See [Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens](#) in Special Instructions for multiple collections.

Forms

If not ordering electronically, complete, print, and send an [Inborn Errors of Metabolism Test Request](#) (T798) with the specimen.

Urine Preservative Collection Options

Note: The addition of preservative or application of temperature controls **must occur within 4 hours of completion** of the collection.

Ambient	No
Refrigerate	OK
Frozen	OK
50% Acetic Acid	No
Boric Acid	No
Diazolidinyl Urea	No
6M Hydrochloric Acid	No
6M Nitric Acid	No
Sodium Carbonate	No
Thymol	No
Toluene	Preferred

Specimen Minimum Volume

1 mL

Reject Due To

All specimens will be evaluated at Mayo Clinic Laboratories for test suitability.

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Urine	Frozen (preferred)	70 days	
	Refrigerated	14 days	

Clinical and Interpretive

Clinical Information

Cystinuria is an inborn error of metabolism resulting from poor absorption and reabsorption of the amino acid cystine in the intestines and kidneys. This leads to an accumulation of poorly soluble cystine in the urine and results in the production of kidney stones (urolithiasis). Symptoms may include acute episodes of abdominal or lower back pain, presence of blood in the urine (hematuria), and recurrent episodes of kidney stones may result in frequent urinary tract infections, which may ultimately result in renal insufficiency. The combined incidence of cystinuria has been estimated to be 1 in 7000.

Cystinuria is an autosomal recessive disease, but some heterozygous carriers have an autosomal dominant, incomplete penetrance appearance with elevated, but typically nondisease causing, urinary cystine excretion. Cystinuria is caused by variants in genes, *SLC3A1* on chromosome 2p and *SLC7A9* on chromosome 19q. Initially, the disease was classified into subtypes I, II, and III (type II and III are also referred as nontype-I) based on the amount of urinary cystine excreted in heterozygous parental specimens. A new classification system has been

proposed to distinguish the various forms of cystinuria: type A, due to variants in the *SLC3A1* gene; type B, due to variants in the *SLC7A9* gene; and type AB, due to 1 variant in each *SLC3A1* and *SLC7A9* gene.

Reference Values**CYSTINE**

3-15 years: 11-53 mcmol/24 hours

> or =16 years: 28-115 mcmol/24 hours

LYSINE

3-15 years: 19-140 mcmol/24 hours

> or =16 years: 32-290 mcmol/24 hours

ORNITHINE

3-15 years: 3-16 mcmol/24 hours

> or =16 years: 5-70 mcmol/24 hours

ARGININE

3-15 years: 10-25 mcmol/24 hours

> or =16 years: 13-64 mcmol/24 hours

Conversion Formulas:

Result in mcmol/24 hours x 0.24=result in mg/24 hours

Result in mg/24 hours x 4.17=result in mcmol/24 hours

Interpretation

Homozygotes or compound heterozygotes with cystinuria excrete large amounts of cystine in urine, but the amount varies markedly. Urinary excretion of other dibasic amino acids (arginine, lysine, and ornithine) is also typically elevated. Plasma concentrations are generally normal or slightly decreased.

Individuals who are homozygous and heterozygous for nontype I cystinuria can be distinguished by the pattern of urinary amino acids excretion: homozygous individuals secrete large amounts of cystine and all 3 dibasic amino acids, whereas heterozygous individuals secrete more lysine and cystine than arginine and ornithine.

Cautions

No significant cautionary statements

Clinical Reference

1. Saravakos P, Kokkinou V, Giannatos E: Cystinuria. Current Diagnosis and Management. Urology April 2014;4(83):693-699

2. Palacin, M, Goodyer, P, Nunes V, Gasparini P: Cystinuria. Edited by D Valle, AL Beaudet, B Vogelstein, et al. New York, McGraw-Hill, 2014. Accessed 6 July 2016. Available at

<http://ommbid.mhmedical.com/content.aspx?sectionid=62653976&bookid=971&Resultclick=2>

Performance

Method Description

Quantitative analysis of the amino acids (AA) cystine, lysine, arginine, and ornithine is performed by liquid chromatography-tandem mass spectrometry (LC-MS/MS) by labeling amino acids present in urine with aTRAQ Reagent 121. Samples are dried and reconstituted with aTRAQ Reagent 113-labeled Standard Mix. Amino acids are separated and detected by LC-MS/MS. The concentrations of amino acids are established by comparison of their ion intensity (121-labeled amino acids) to that of their respective internal standards (113-labeled amino acids). Chromatography is performed using a C18 (150x4.6mm) column and total analysis time is 18 minutes. (Lacey JM, Casetta B, Daniels SB, et al: Quantitation in plasma, urine and CSF by ltraq reagent amino acid analysis kit and MS-MS. J Am Soc Mass Spec 2008;19[5]:S97)

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

3 to 5 days

Specimen Retention Time

2 weeks

Performing Laboratory Location

Rochester

Fees and Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Mayo Clinic in a manner consistent with CLIA requirements. This test has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

82136

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
CYSQN	Cystinuria Profile, QN, 24 hour	In Process



Result ID	Test Result Name	Result LOINC Value
TM38	Collection Duration	13362-9
VL36	Urine Volume	3167-4
3583	Cystine	13966-7
3589	Lysine	25464-9
3590	Ornithine	25491-2
3591	Arginine	25322-9
50447	Reviewed By	18771-6